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INSTRUCTIONS FOR USE MENSTRUAL SPONGES

This set contains two menstrual sponges sourced from the Mediterranean. They belong to the Levantine (Hipposporgia gossypina) genus and are harvested by hand in a sustainable manner. The sponges are cleaned and softly bleached. As this is a natural product, the shape of the sponges may vary.

PREPARATION

Before using the natural sponges for the first time, we recommend neutralising and cleaning them in vinegar water. To do this, soak overnight in household vinegar in a ratio of 1:2, or in vinegar essence in a ratio of 1:10.

If the menstrual sponge appears to be too big for you, you can cut it to your personal preference. Please first moisten the sponge to see what the size is like when wet and compressed.

If desired, you can also sew a little thread to the sponge to facilitate easier use.

Be sure to wash your hands before using the sponges. To simplify insertion, hold the sponge under clear water and then squeeze it well to ensure maximum absorbency.

The easiest way to insert the sponge is, for instance, while squatting or sitting on the toilet. Grasp the sponge on the sides, using two fingers and press it together. Now insert the narrow side into the vagina with one finger, similar to the way you would do it with a tampon. The sponge will adapt to the natural shape.

REMOVAL

Removing the sponge is easier when you squat. Apply pressure to the sponge from the inside so that it comes further forward and is easier to reach.

The menstrual sponge is easier to remove when fully soaked. If you have problems removing it, a vaginal douche can help.

Please do not forget to remove the sponge at the end of the period.

WEARING DURATION

We recommended changing the sponge at regular intervals. The duration varies, depending on your period and should not be longer than maximally 8 hours. In case of heavier bleeding, we recommend changing the sponge every 2-4 hours.